

Milliken™

MILLIKEN HEALTHCARE PRODUCTS, LLC











BEST CLINICAL PRACTICES

BEST CLINICAL PRACTICES



Milliken Healthcare Products, LLC TRITEC™, ULTRA and ASSIST dressings have the versatility to be used across a wide range of wound types and are effective from low to highly exuding wounds. Every TRITEC, ULTRA and ASSIST dressing utilizes Active Fluid Management® Technology, a three part design that combines **two layers** of high performance fabric using a **micro-knit process**. This creates a powerful moisture transfer mechanism that pulls away and traps excess exudate while leaving an optimal amount of moisture at the wound bed.


WOUND TYPE	Shallow Pressure Ulcer	Deep Pressure Ulcer	Diabetic Ulcer	Venous Ulcer	Arterial Ulcer	Partial Thickness Burn	Skin Graft / LSE	Surgical Incision
WOUND DESCRIPTION	Superficial, often appearing as an abrasion, blister, or shallow skin loss	Full thickness ulceration that may extend to fascia	Ulcer of the foot caused by pressure or stress, typically accompanied by neuropathy or loss of sensation	Ulceration of the lower leg, caused by venous insufficiency, typically accompanied by edema and heavy exudate	A lower extremity ulcer caused by impairment of arterial blood flow, often resulting in tissue ischemia	Burn that extends through the epidermis and into the dermis	Healthy skin or tissue-engineered skin used in burn and chronic wound treatment	A wound or cut as a result of a surgical procedure
WOUND VISUAL								
PRIMARY DRESSING	TRITEC Silver 4x5, 6x6 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5	TRITEC Silver 4x5, 6x6 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5	TRITEC Silver 4x5, 1x24 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5	TRITEC Silver 4x5, 4x48 TRITEC 4x5, 4x48 ULTRA Silver 4x5 ULTRA 4x5	TRITEC Silver 4x5, 4x48 TRITEC 4x5, 4x48 ULTRA Silver 4x5 ULTRA 4x5	ASSIST Silver 4x48, 6x6, 8x20 TRITEC Silver 4x5, 4x48 TRITEC 4x5, 4x48 ULTRA Silver 4x5 ULTRA 4x5	ASSIST Silver 4x48, 6x6, 8x20 TRITEC Silver 4x5 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5	TRITEC Silver 4x5, 1x24 TRITEC 4x5 ULTRA Silver 4x5 ULTRA 4x5
SECONDARY DRESSING	ULTRA 4x5, gauze or equivalent absorptive dressing Wound veil as needed	ULTRA 4x5, gauze or equivalent absorptive dressing Wound filler as needed	ULTRA 4x5, gauze or equivalent absorptive dressing	ULTRA 4x5, gauze or equivalent absorptive dressing May be used under compression	ULTRA 4x5, gauze or equivalent absorptive dressing	ULTRA 4x5, gauze or equivalent absorptive dressing Wound veil suggested (Wound veil not needed with ASSIST)	ULTRA 4x5, gauze or equivalent absorptive dressing Wound veil suggested (Wound veil not needed with ASSIST)	ULTRA 4x5, gauze or equivalent absorptive dressing

CHANGE SECONDARY DRESSING AS NEEDED

THE ANTIMICROBIAL PROPERTIES OF TRITEC SILVER AND ASSIST SILVER DRESSINGS ARE PROVEN TO LAST UP TO 7 DAYS
CONSIDER TRITEC SILVER, ULTRA SILVER OR ASSIST SILVER PRODUCTS WHEN MICROBIAL CONTAMINATION IS A CONCERN

The information provided herein is not to be construed as the practice of medicine or substituted for the independent medical judgment of a patient's treating physician. This information, including but not limited to suggestions for product usage, product selection and suggested use is based on independent generalizations and does not consider the unique characteristics of an individual's wound. Each patient's physician shall remain solely responsible for assessing the severity of patient wounds, determining the appropriate treatment and managing treatment of the wound.

Please see product insert for complete list of indications and instructions for use.

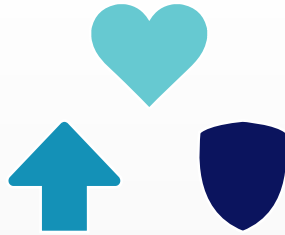

pull preserve protect

Pull exudate away from the wound.
Preserve optimal moisture.
Protect healthy skin.

medproducts@milliken.com | healthcare.milliken.com | 1-866-491-6556