



QuickVue In-Line®

Strep A



Test and Treat Today!™

Rapid Diagnosis and Treatment

- Results in 5 minutes, or less
- Most patients tested can be diagnosed and treated in just one visit.

Excellent Performance*

- 92% sensitivity
- 99% specificity
- 96% overall agreement with SBA culture

Easy to Use

- One easy step via In-Line extraction.
- Two-color endpoint for easy to read and interpret results.
- Single reagent method allows for less chance of operator error.


Convenient Packaging

- All components necessary to perform a test are contained in the kit, including swabs.
- Room temperature storage (24 months from date of manufacture).
- Built-in Positive and Negative controls confirm the test has performed properly.
- Positive and Negative external controls provided in the kit.
- Individually packaged for convenient single-use testing and disposal.


*In-house study conducted at Quidel. Refer to Package Insert for additional performance claims.

Procedure


MUST use the swabs provided in the kit

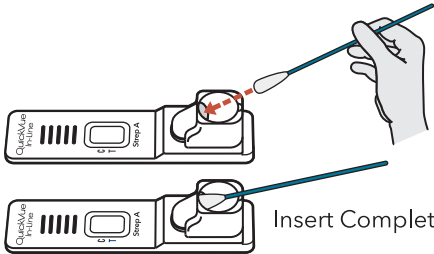


Insert Completely

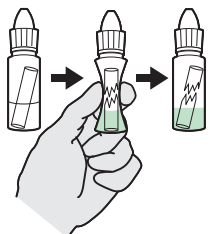


Squeeze **ONCE** to break.



Shake

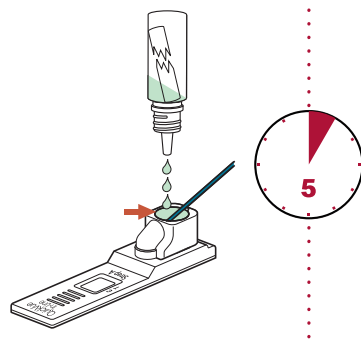

Quickly Fill to Rim

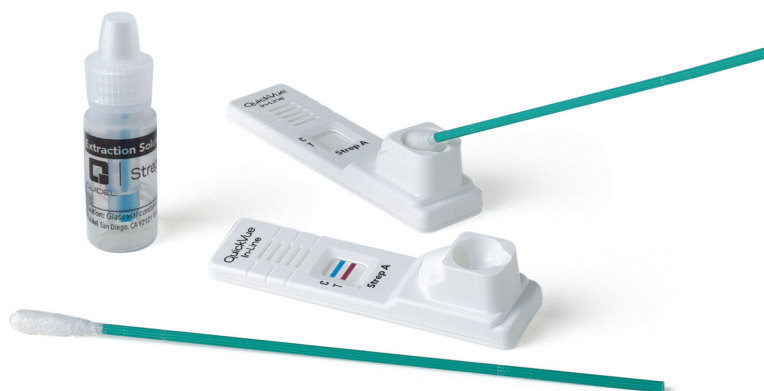

QuickVue In-Line Strep A Test – 25 Test Kit: Catalog #00343
QuickVue In-Line Strep A Control Set – Catalog #00345
QuickVue In-Line Strep A Swab Pack – 75 Swabs: Catalog #00347

SL8005 | SS0034300EN00 (03/20)

