

Strep A FIA



Rapid Strep A . . . brought to you by Sofia

Enhanced performance and productivity

- Enhanced assay sensitivity
- WALK AWAY and READ NOW modes provide flexible workflow options
- Objective results in 5 minutes


Record keeping flexibility

- Barcode scanner captures user and patient IDs
- Digital record of QC, calibration and patient results
- Transmission of data to EMR via LIS/Middleware


Virena® surveillance and remote data management capabilities

- Automatically captures patient de-identified results
- Generates, trends and maps disease prevalence through your organization year-round
- User can create reports and files for in-depth analysis and actionable response


Procedure

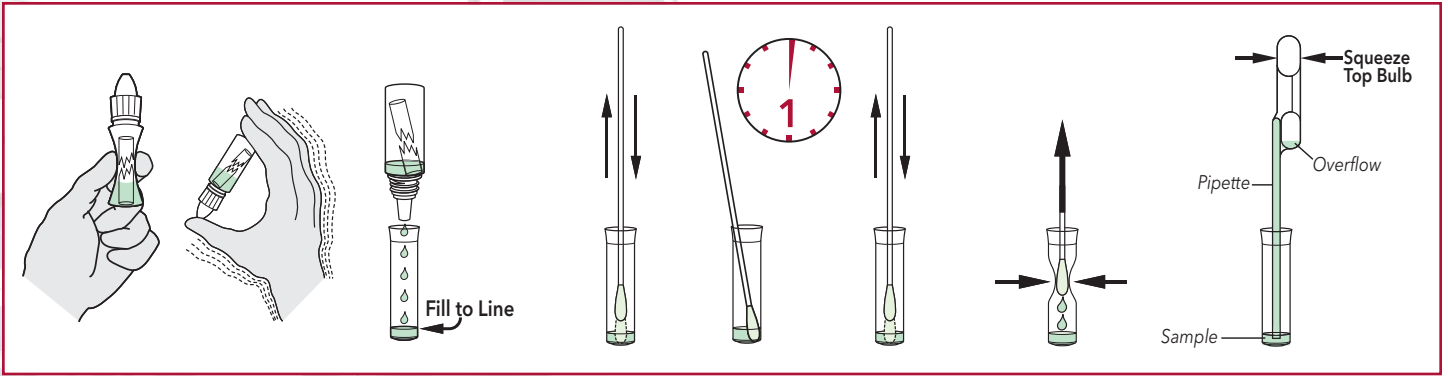

Walk Away Mode – Walk away and multitask

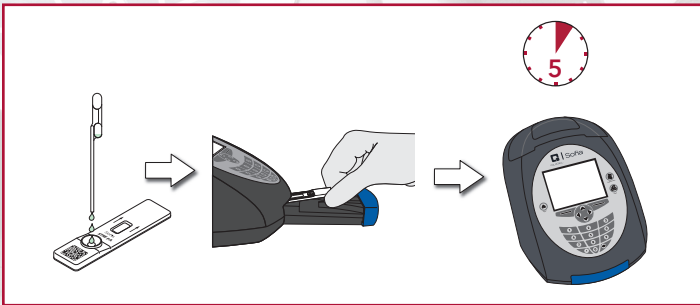

Read Now Mode – Batch multiple samples per hour

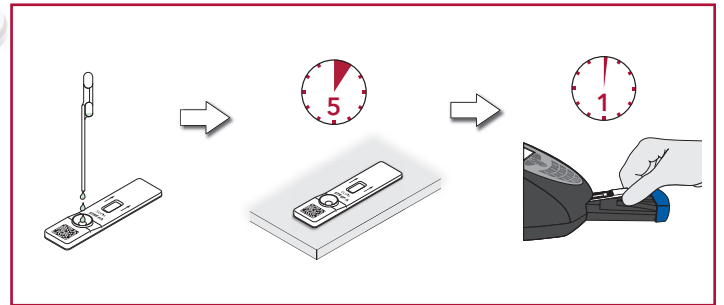

Sofia Strep A FIA

- **Moderately complex**
- Accurate detection with throat swab samples
- Excellent performance demonstrated at a multi-site study*
- Room temperature storage
- 24-month shelf life from date of manufacture
- Compatible with two transport media (see Package Insert for details)

Sofia Strep A FIA Results vs. Culture*

	Culture			Sensitivity	90.6% (116/128)
	Pos	Neg	Total		
Sofia Pos	116	24	140	Specificity	96.1% (584/608)
Sofia Neg	12	584	596		
Total	128	608	736		

Source: Multi-center clinical field study conducted in eight distinct sites in the U.S. and two in Australia. Samples included 736 patients.

*Refer to the Package Insert for additional performance claims.