Professional Procedure Guide

A1CNow+[®]
Professional
Multi-Test HbA1c System


Introduction

- Run the test with all parts of the test kit at the same temperature within the specified range.
- If the kit has recently been at high temperatures (above 82°F) or in the refrigerator, keep the kit at room temperature for at least one hour before use.
- Avoid running the test in direct sunlight, on hot or cold surfaces, or near sources of heat or cold.
- Quality control materials should be used to confirm the test kit is working properly. Refer to the product insert for information on when to run controls.


5 Mix

- Shake vigorously 6-8 times. This will mix the blood with the solution.
- Stand shaker on table while preparing cartridge.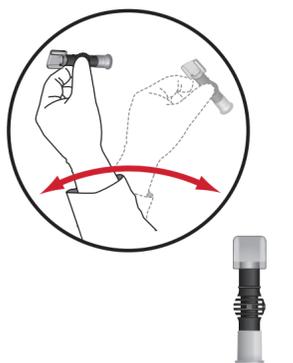

Match Lot Numbers


Use analyzer only with the materials included in the original kit. The analyzer will expire after the programmed number of tests have been run. If another test cartridge is inserted, the analyzer will display "00 TL."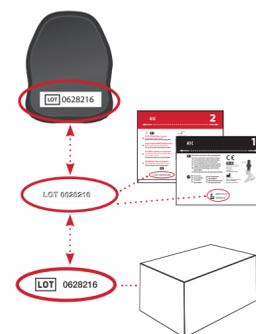

6 Open Foil Cartridge Pouch*

Tear foil pouch open at the notches on the sides.

Caution: Do NOT open foil pouch until this step. Use within 2 minutes of opening. If foil pouch is damaged, do not use.

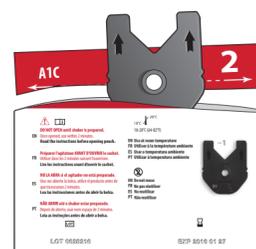1 Open Plastic Shaker Pouch*

Tear plastic pouch open at the perforation line.

2 Collect Blood

Fingerstick method

- Use your own lancet device to draw blood.
- Gently touch blood drop to fill.


OR

Venous draw method

- Mix blood well before testing.
- Collect blood from a slide.

3 Verify Blood Amount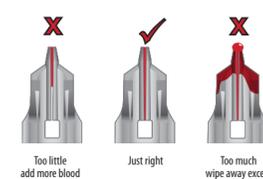

4 Insert Blood Collector

Fully insert blood collector into shaker body. Twisting motion helps.

Note: All instances of the box or pouches in this insert are representative ONLY. Please refer to the packaging included with your specific kit.

pts Diagnostics

* Pouch illustrations may vary.


7 Insert Cartridge

- "Click" test cartridge into place.
- Analyzer and test cartridge codes must match.
- If codes do not match, call Customer Service at 1-877-870-5610.

8 Prepare Shaker

Remove shaker base.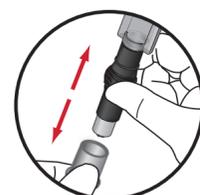

9 Dispense Sample into Cartridge

- Ensure analyzer is on level surface.
- Push down completely to dispense diluted sample. Remove quickly.

Do not handle analyzer again until test is complete!

10 5 Minutes to Results

- This result cycle remains displayed for 15 minutes or until the next test cartridge is inserted.
- If "QCOK" is not displayed, please contact Customer Service or consult the troubleshooting section.

11 Dispose Cartridge / Save Analyzer

Dispose of in accordance with your biohazard procedures. Analyzer may be re-used until tests have been deleted. See "Storage and Handling" for disinfection and storage instructions.

NOTE: To run another test, use a new blood collector, shaker, and test cartridge from the same kit and return to Step 1.

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A1CNow+[®]
Professional
Système multitest HbA1c

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