


TITAN™ XL Irrigation Container

Helpful Hints

Refer to product labeling for full prescribing information. Available in 2,000 mL and 3,000 mL, both come in 4 per case.

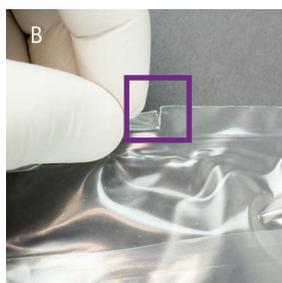


REMOVING OVERWRAP

Using strict aseptic technique, tear overwrap starting from tear notches at either end. Inspect for leaks or particulates before adding medications or spiking.

ADDING MEDICATIONS

Step 1: If adding medication, remove foil seal from CLEAR add port.
Step 2: Puncture resealable port using 19 to 22 gauge needle. Mix thoroughly. Medication port must be swabbed with disinfecting agent before re-puncturing.

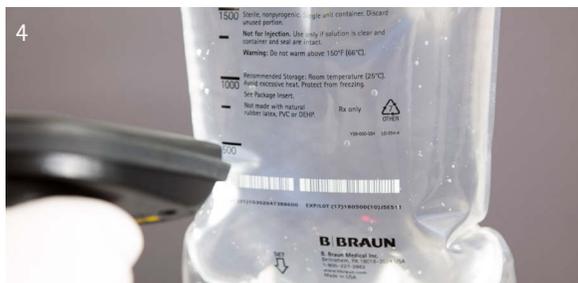

SPIKING

Step 3: Remove foil seal from GREEN set port. Pass the administration set spike through the GREEN set port with integrated "O" ring. Stop advancing the spike once the spike shoulder meets the GREEN set port.

CAUTION: Insertion of the spike past the shoulder could result in leakage.

SCANNING

Step 4: Lay the container on a flat surface or hang the container on an IV pole and scan at a 45°–90° degree angle, 2"–6" from the container.

B. Braun Medical Inc. | Bethlehem, PA
1-800-227-2862 | bbraunusa.com