


INSTRUCTIONS FOR

Sterile Vial Adapters (#20015 & 20016)

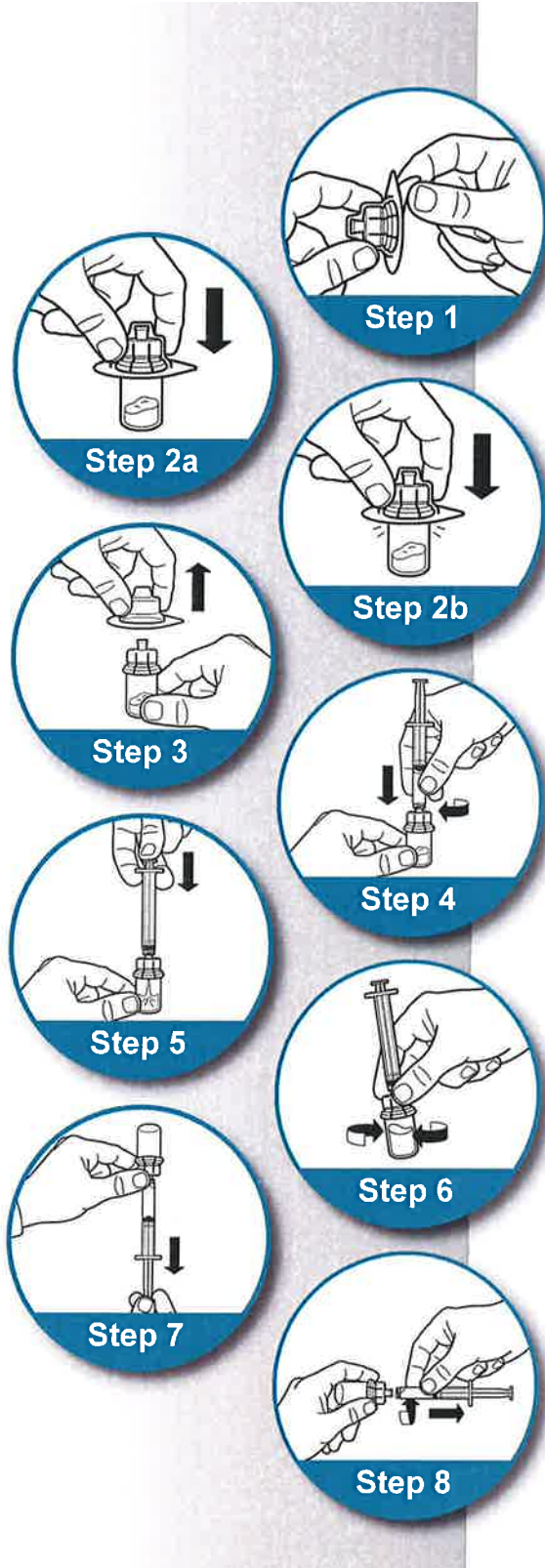

Before handling the components, wash your hands with soap and hot water.

Place the package on a clean, flat surface.

Remove the Flip-Off® button from the drug product vial.

Disinfect the injection site with an alcohol pad.

Step 1:

Remove the cover from the vial adapter package.

Do not remove the vial adapter from the blister package.

Step 2:

Attach the adapter to the vial; use the blister pack to handle the adapter. Seat the adapter on the vial by pushing down until the spike penetrates the rubber stopper and the adapter snaps in place.

Step 3:

Remove and discard the blister package.

Step 4:

Attach the diluent prefilled syringe to the adapter.

Step 5:

Transfer the diluent to the vial by slowly pushing the plunger all the way in.

Step 6:

Gently swirl the vial to make sure the product is thoroughly mixed.

Step 7:

Withdraw the reconstituted drug from the vial into the syringe.

Step 8:

Remove the syringe from the adapter.

The drug is now ready for administration.

Follow normal safety practices to administer the drug.

Call Free 1.800.848.1633 • Fax Free 1.800.447.2923

Web: GoHCL.com • Email: hcl@GoHCL.com