

**GEIGER DISPOSABLE THERMAL CAUTERY ELECTRODES: PART
NO. 210T, 212, 214**

CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN OR HEALTH CARE PROFESSIONAL.

CAUTION: THIS DEVICE IS SUPPLIED NON-STERILE.

CAUTION: DO NOT USE THIS DEVICE IN THE PRESENCE OF FLAMMABLE MATERIALS.

WARNING: READ ALL THERMAL CAUTERY DEVICE INSTRUCTIONS PRIOR TO USING THIS DEVICE.

FOR USE WITH 250 and 202 HANDPIECES.

Inspection and Warranty

Inspect the electrode upon receipt. If damage has occurred during shipment, do not use and notify Geiger immediately.

Instructions for Use

1. Insert the electrode into the corresponding holes of the thermal cautery handpiece. The electrode must be fully and securely inserted.
2. At the completion of the procedure, remove the electrode from the handpiece and dispose of properly.

SINGLE USE ONLY – DO NOT STERILIZE OR REUSE.

Note: Make sure that the cautery electrode is making satisfactory contact with the walls of the handpiece receptacle holes. You may want to slightly bend the electrode shafts to increase the contact area if the electrode is loose.