

AIMSTEP® H.PYLORI

AimStep® H. Pylori is a rapid test for the qualitative detection of antibodies specific to Helicobacter pylori in serum, plasma, or whole blood.

SIMPLE PROCEDURE

SERUM, PLASMA & VENIPUNCTURE WHOLE BLOOD







Fill capillary tube to "black line" $(10\mu L)$.

Add 2 to 3 drops of Developer Solution to Sample Well.

Start timer. Wait 10 minutes.

INCLUDES:



91.6% AGREEMENT WITH BIOPSY **HISTOLOGY**

STORE AT 2° TO 30°C




POSITIVE AND NEGATIVE CONTROLS INCLUDED



SINGLE REAGENT **PROCEDURE**



FINGER SAMPLE COLLECTION

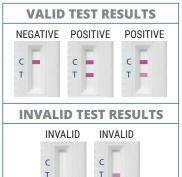

FINGERSTICK CAPILLARY WHOLE BLOOD





STEP 1

Fill capillary tube to "red line" (25µL).

STEP 2

Add 2 to 3 drops of Developer Solution to Sample Well.

Start timer. Wait 10 minutes.

READING RESULTS

VALID TEST RESULTS			
NEGATIVE	POSITIVE		POSITIVE
C -	C T		C — T —
INVALID TEST RESULTS			
INVALID		INVALID	
C T		C T	

Look at these great features:

- Collect the specimen that you want: serum, plasma, venipuncture and fingerstick whole blood
- procedures available via email or fax; just add your name
- Priced to meet your budget requirements

CPT Code: 86318QW

THE TESTS YOU WANT. THE ACCURACY YOU NEED!

www.germainelabs.com





800.854.8446 210.692.4192

210.692.4198 Fax

DISTRIBUTED BY: See Package Insert for complete details.