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ez adjust  
aluminum crutches  
Push-Button/Adjustable Fully Assembled

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10432-1  
Grande  
taille

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### CAUTIONS:

- Use after proper training under the direction of your Doctor or Physical Therapist.
- Replace Crutch Tips if worn.
- Use extra caution on wet and slippery surfaces.

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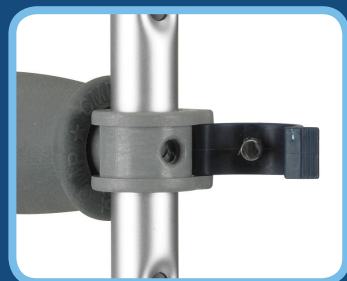
## For maximum fit and assembly, see instructions below:

1. Height: Adjust height by pressing both push-buttons, rotating foot-piece, and pulling down until desired patient height is obtained. To help in doing this, the foot-piece has a circular ring marked to indicate location of push-button. Rotate foot-piece again until both push buttons snap back into place.

2. IMPORTANT SAFETY FEATURE WHEN ADJUSTING HANDLE HEIGHT:

Unlock both blue euro-style clips on hand grip by using thumbs and pressing outward on thumb tab. Set hand grip by placing underarm pad under armpit and sliding hand grip so that the elbow has a 10° bend. When correct position is achieved, lock euro-style clips in place by inserting into positioning hole and locking push tab.

3. Safety Check: With crutches positioned per diagram below, confirm patient can lean forward comfortably while resting on the underarm supports. Patient should not have to stand uncomfortably erect. Ensure push-buttons on bottom adjustment tube are firmly in place and euro-style clips are in the locked position.

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