# **SafeWear**<sup>™</sup>

HIGH PERFORMANCE LAB COAT™





- Thoughtful designs offer ease of movement, as well as secure closure and easy removal.
- Premium materials provide superior protection, reliability and durability.
- Lightweight, soft, supple, fabrics keep wearers cool and comfortable.
- SafeWear™ protective apparel complies with recommended OSHA best practices.

## RECOMMENDED USE

 Procedure SMS Lab Coat designed to protect health professionals and patients for cross-contamination.
 Single use and disposable gowns made of polypropylene.
 Material meets AAMI/PB70 Level 2 requirements for impact penetration and hydrostatic pressure. Latex-free.

### PRODUCT DETAILS

Product Name: Medicom SafeWear™ High

Performance Lab Coat™

Product Type: Single-use, non-sterile
Color: See Appendix 1
Sizes: See Appendix 1

Quantity per Bag: 12 Country of Origin: China

## **DONNING INSTRUCTIONS**

- Perform hand hygiene before removing item from package.
- Select appropriate garment for intended use (length, coverage, fluid resistance, etc.)
- Fasten correctly: tied at neck and waist to ensure comfortable fit, adequate coverage and freedom of movement.
- Protective apparel should not be worn in restrooms, personal offices or break rooms.

### **DOFFING INSTRUCTIONS**

- o Remove apparel immediately after use. Do not re-use.
- Perform proper hand hygiene after removing apparel to prevent cross-contamination.
- Turn garment inside out when removing to keep infectious material inside.
- Designate a safe receptacle in which to dispose of protective apparel after use.



## CERTIFICATION & STANDARDS

ISO 13485 certified manufacturer. Class 1 Medical Device.

Health Canada registered medical device manufacturer.

# STORAGE CONDITIONS

These gowns should be disposed of after each use. Store under ambient temperature and dry condition.









|   | GOWN SPECIFICATIONS                     |          |             |             |              |              |                |               |  |  |  |  |
|---|---|----------|-------------|-------------|--------------|--------------|----------------|---------------|--|--|--|--|
| CHARACTERISTIC SPECIFICATION                              |   |          |             |             |              |              |                |               |  |  |  |  |
|   | Material Composition                    | Spunbond | Meltdown Sp | ounbond (SM | S) polypropy | lene         |                |               |  |  |  |  |
| Description Coat made of non-wa<br>and cuffs, and two sli |   |          |             |             | th front and | snap buttons | . Features: kr | nitted collar |  |  |  |  |
|   | 2 . (C                                  |          |             | С           | imension (cr | n)           |                |               |  |  |  |  |
|   | Part of Gown                            | X-Small  | Small       | Medium      | Large        | X-Large      | 2X-Large       | 3X-Large      |  |  |  |  |
| A   | Coat Length                             | 94       | 100         | 112         | 114          | 116          | 120            | 120           |  |  |  |  |
|   | Underarm Width                          | 50       | 55          | 60          | 67           | 70           | 76             | 90            |  |  |  |  |
| С   | Sleeve Length*                          | 50       | 52          | 54          | 57           | 60           | 60             | 64            |  |  |  |  |
| D   | Arm Height                              | 27       | 28          | 29          | 30           | 31           | 33             | 36            |  |  |  |  |
| E   | Shoulder Width                          | 16       | 18          | 18          | 20           | 23           | 26             | 29            |  |  |  |  |
| F   | Knitted Cuff Length                     | 6        | 6           | 6           | 6            | 6            | 6              | 6             |  |  |  |  |
| L   | Sleeve Opening Width (Stretched)        | 15       | 15          | 15          | 15           | 15           | 15             | 17            |  |  |  |  |
| М   | Knitted Cuff Width                      | 6        | 6           | 6           | 6            | 6            | 6              | 6             |  |  |  |  |
| G   | Button Spacing                          | 15.5     | 15.5        | 15.5        | 15.5         | 15.5         | 15.5           | 15.5          |  |  |  |  |
|   | Lower Front Pocket - Width              | 18       | 18          | 18          | 18           | 18           | 18             | 18            |  |  |  |  |
| н   | Lower Front Pocket - Height             | 18       | 18          | 18          | 18           | 18           | 18             | 18            |  |  |  |  |
|   | Upper Pocket - Width                    | 14       | 14          | 14          | 14           | 14           | 14             | 14            |  |  |  |  |
| J   | Upper Pocket - Height                   | 14       | 14          | 14          | 14           | 14           | 14             | 14            |  |  |  |  |
| N   | Upper Pocket Distance from Opening Edge | 5        | 6           | 9           | 9            | 9            | 10             | 11            |  |  |  |  |
| Р   | Lower Pocket Distance from Opening Edge | 5        | 5           | 6           | 6            | 8            | 8              | 10            |  |  |  |  |
| S   | Distance between Upper and Lower Pocket | 23       | 27          | 30          | 30           | 31           | 31             | 31            |  |  |  |  |
| 0   | Lower Pocket Distance from Bottom Edge  | 16       | 18          | 26          | 27           | 32           | 29             | 33            |  |  |  |  |
| R   | Back Slit Height                        | 30       | 30          | 30          | 30           | 30           | 30             | 30            |  |  |  |  |
| Q   | Back Panel Width                        | 40       | 43          | 48          | 55           | 58           | 64             | 78            |  |  |  |  |
|   | Number of Buttons                       | 5        | 5           | 5           | 5            | 5            | 5              | 5             |  |  |  |  |



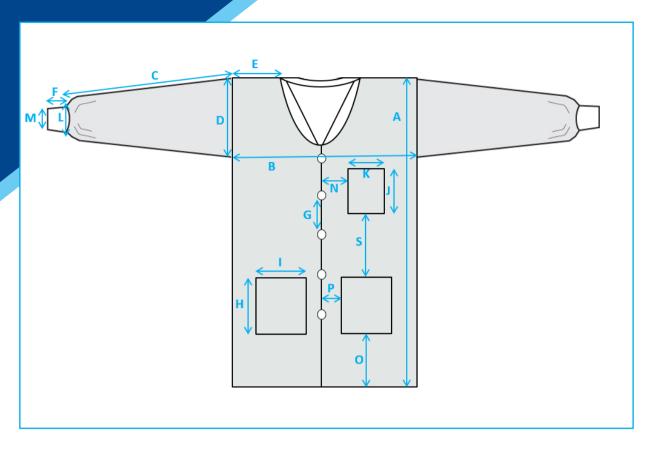

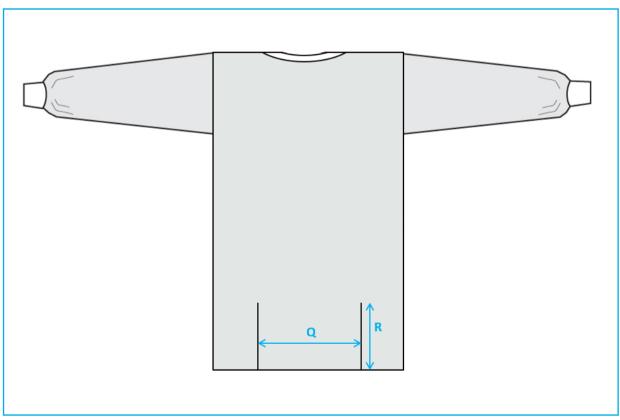



|           | Appendix 1 (List of items)                       |  |         |           |   |             |                              |                           |  |  |  |
|-----------|--|--|---------|-----------|---|-------------|------------------------------|---------------------------|--|--|--|
| Item Code | Description                                      | Packaging<br>Information   | Size    | Color     | Specific<br>Attribute                               | Treatment   | Marketed<br>in USA as        | Marketed in<br>Canada as  |  |  |  |
| 8107-A    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Small   | Deep Blue | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8107-B    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Medium  | Deep Blue | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8107-C    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Large   | Deep Blue | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8107-D    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | X-Large | Deep Blue | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |

APP1 01.02-30EN\_1M Date: 12-Aug-2021 Page 1 / 7

|           | Appendix 1 (List of items)                       |  |          |                |   |             |                              |                           |  |  |  |
|-----------|--|--|----------|----------------|---|-------------|------------------------------|---------------------------|--|--|--|
| Item Code | Description                                      | Packaging<br>Information   | Size     | Color          | Specific<br>Attribute                               | Treatment   | Marketed in USA as           | Marketed in Canada as     |  |  |  |
| 8107-E    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | 2X-Large | Deep Blue      | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8107-F    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | 3X-Large | Deep Blue      | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8108-A    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Small    | Pretty<br>Pink | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8108-B    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Medium   | Pretty<br>Pink | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8108-C    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Large    | Pretty<br>Pink | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |

APP1 01.02-30EN\_1M Date: 12-Aug-2021 Page 2 / 7

| Appendix 1 (List of items) |  |  |         |                |   |             |                              |                           |  |  |
|----------------------------|--|--|---------|----------------|---|-------------|------------------------------|---------------------------|--|--|
| Item Code                  | Description                                      | Packaging<br>Information   | Size    | Color          | Specific<br>Attribute                               | Treatment   | Marketed<br>in USA as        | Marketed in<br>Canada as  |  |  |
| 8108-D                     | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | X-Large | Pretty<br>Pink | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |
| 8109-A                     | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Small   | Plum<br>Purple | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |
| 8109-B                     | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Medium  | Plum<br>Purple | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |
| 8109-C                     | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Large   | Plum<br>Purple | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |
| 8109-D                     | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | X-Large | Plum<br>Purple | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |

APP1 01.02-30EN\_1M Date: 12-Aug-2021 Page 3 / 7

|           | Appendix 1 (List of items)                       |  |         |                |   |             |                              |                           |  |  |  |
|-----------|--|--|---------|----------------|---|-------------|------------------------------|---------------------------|--|--|--|
| Item Code | Description                                      | Packaging<br>Information   | Size    | Color          | Specific<br>Attribute                               | Treatment   | Marketed in USA as           | Marketed in<br>Canada as  |  |  |  |
| 8110-A    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Small   | White<br>Frost | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8110-B    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Medium  | White<br>Frost | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8110-C    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Large   | White<br>Frost | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8110-D    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | X-Large | White<br>Frost | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8112-A    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Small   | Soft Blue      | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |

APP1 01.02-30EN\_1M Date: 12-Aug-2021 Page 4 / 7

|           | Appendix 1 (List of items)                       |  |         |                  |   |             |                              |                           |  |  |  |
|-----------|--|--|---------|------------------|---|-------------|------------------------------|---------------------------|--|--|--|
| Item Code | Description                                      | Packaging<br>Information   | Size    | Color            | Specific<br>Attribute                               | Treatment   | Marketed in USA as           | Marketed in<br>Canada as  |  |  |  |
| 8112-B    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Medium  | Soft Blue        | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8112-C    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Large   | Soft Blue        | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8112-D    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | X-Large | Soft Blue        | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8117-A    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Small   | Tropical<br>Teal | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8117-B    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Medium  | Tropical<br>Teal | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |

APP1 01.02-30EN\_1M Date: 12-Aug-2021 Page 5 / 7

|           | Appendix 1 (List of items)                       |  |         |                  |   |             |                              |                           |  |  |  |
|-----------|--|--|---------|------------------|---|-------------|------------------------------|---------------------------|--|--|--|
| Item Code | Description                                      | Packaging<br>Information   | Size    | Color            | Specific<br>Attribute                               | Treatment   | Marketed in USA as           | Marketed in Canada as     |  |  |  |
| 8117-C    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Large   | Tropical<br>Teal | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8117-D    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | X-Large | Tropical<br>Teal | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8119-A    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Small   | Jet Black        | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8119-B    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Medium  | Jet Black        | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |
| 8119-C    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | Large   | Jet Black        | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |

APP1 01.02-30EN\_1M Date: 12-Aug-2021 Page 6 / 7

|           | Appendix 1 (List of items)                       |  |         |           |   |             |                              |                           |  |  |  |
|-----------|--|--|---------|-----------|---|-------------|------------------------------|---------------------------|--|--|--|
| Item Code | Description                                      | Packaging<br>Information   | Size    | Color     | Specific<br>Attribute                               | Treatment   | Marketed in USA as           | Marketed in<br>Canada as  |  |  |  |
| 8119-D    | Medicom SafeWear<br>High Performance Lab<br>Coat | 12 Coats/Bag, 5 Bags/Case,<br>60 Coats/CaseA<br>cardboard strip is present<br>on the top of the case | X-Large | Jet Black | Matching<br>Sewing<br>Thread<br>Color, 3<br>Pockets | Non-Sterile | Medical<br>Device<br>Class 1 | Medical<br>Device Class 1 |  |  |  |

AMD Medicom Inc. | 2555 Chemin de l'Aviation, Montreal, Quebec, Canada H9P 2Z2

APP1 01.02-30EN\_1M Date: 12-Aug-2021 Page 7 / 7