

# THERE'S A LIFE BEHIND EVERY TEST

## COVID-19 & FLU A+B WITH ONE ASSAY

The BD Veritor™ Plus System now allows more options on a single assay, allowing you to test for **COVID-19** and **Flu A+B** with just one sample. With multiple testing modes and workflow efficiencies, the Analyzer is more versatile than ever.

One sample. Three tests. Reliable results.  
Combo testing for **COVID-19\*** and **Flu A+B** is here.



 **BD Veritor™ Plus**  
Analyzer

- \*These products have not been FDA cleared or approved; but have been authorized by FDA under EUA for use by authorized laboratories
- The BD Veritor™ System for Rapid Detection of SARS-CoV-2 & Flu A+B has been authorized only for the detection of proteins from SARS-CoV-2, influenza A and influenza B, not for any other viruses or pathogens; the BD Veritor™ System for Rapid Detection of SARS-CoV-2 has been authorized only for the detection of proteins from SARS-CoV-2, not for any other viruses or pathogens; and,
- These products are only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.



# Get more answers with every sample.

COVID-19 & FLU A+B

Test for **COVID-19\*** and **Flu A+B** with a single sample, using the 3-in-1 assay from the **BD Veritor™ Plus System**.



## Simplifies the testing process

- May help reduce manual test processing errors with its easy operation and single-button functionality



## Achieves reliable, rapid results

- Displays easy-to-read digital results for COVID-19 and Flu A+B in 15 minutes<sup>1</sup>:
  - » “CoV2: +” for positive; “CoV2: -” for negative
  - » “FLU A: +” for positive; “FLU A: -” for negative
  - » “FLU B: +” for positive; “FLU B: -” for negative
- Provides clear, objective results by correcting for nonspecific binding and detecting positives not recognized by the unaided eye<sup>1</sup>



## Delivers workflow efficiency

- Adapts easily to your workflow by offering 2 operational modes
  - » **Walk Away:** the test device is inserted immediately into the Analyzer so staff can multitask while the sample incubates (15 minutes)
  - » **Analyze Now:** the test device is inserted after the incubation time is complete, allowing batches of samples to be tested (results in seconds)



## Provides result traceability

- Captures or downloads the lot number, patient/specimen ID, operator ID, and test records using the BD Veritor™ InfoWiFi module

**Reference:** 1. BD Veritor™ Plus System for Rapid Detection of SARS-CoV-2 and Flu A+B. Package insert. Becton, Dickinson and Company.

BD Life Sciences, 7 Loveton Circle, Sparks, MD 21152-0999 USA  
800-638-8663

**BDVeritor.com**

BD, the BD Logo and Veritor are trademarks of Becton, Dickinson and Company or its affiliates. © 2021 BD. All rights reserved. 723-US-1220 April 2021

## Clinical performance vs RT-PCR<sup>1</sup>

	PPA	NPA	Results In
<b>SARS-CoV-2</b>	<b>86.7%</b> (95% CI, 75.8%-93.1%)	<b>99.5%</b> (95% CI, 97.4%-99.9%)	
<b>Influenza A</b>	<b>82.7%</b> (95% CI, 74.9%-88.5%)	<b>97.5%</b> (95% CI, 95.7%-98.5%)	
<b>Influenza B</b>	<b>80.7%</b> (95% CI, 70.3%-88.1%)	<b>98.2%</b> (95% CI, 95.7%-99.3%)	

SARS-CoV-2 and Flu A+B results in 15 minutes

NPA=negative percent agreement;  
PPA=positive percent agreement.

\* A negative result does not rule out COVID-19 infection and should not be used as the sole basis for treatment or patient management decisions, including infection control decisions. A negative result should be considered in the context of a patient's recent exposures, history, and the presence of clinical signs and symptoms consistent with COVID-19, and confirmed with a molecular assay, if necessary, for patient management. For additional information, please refer to the HCP fact sheet.

Ordering information	Cat no.	Qty
BD Veritor™ Plus System SARS-CoV-2 & Flu A+B	256088	30 tests
BD Veritor™ Plus Analyzer	256066	1 unit
<b>Other BD Veritor™ products</b>		
BD Veritor™ Plus System SARS-CoV-2	256082	30 tests
BD Veritor™ Plus System Flu A+B CLIA-waived kit	256045	30 tests
BD Veritor™ Plus System Flu A+B (moderately complex)	256041	30 tests
BD Veritor™ Plus System RSV CLIA-waived kit	256038	30 tests
BD Veritor™ Plus System RSV (moderately complex)	256042	30 tests
BD Veritor™ Plus System Group A Strep CLIA-waived kit	256040	30 tests
BD Veritor™ InfoWiFi module	445010	1 unit
USB printer cable	443907	1 unit